Ao

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[Exportar para Excel](#)[Exportar para PDF](#)[Voltar](#)

The warranties pertain to the complete product.
For parts and pieces sold separately, consult
CBEMED.

Registration with the Ministry of Health:
80540449002
Registration with the Ministry of Health:
80540449003
Registration with the Ministry of
Health:80540440008
Registration with the Ministry of Health:
80540440009

REGULATION
• Directive 93/42 / EEC as amended by Directive
2007/47 / EC
• Directive 93/42 / CEE amended by Directive
2007/47 / EC
• ISO 13485

All warranties are valid from the date of actual
delivery of the products.
For greater agility and in order to avoid improper
freight, in the case of more careful analysis,
CBEMED indicates that the product is sent by the
customer and the manufacturing defect is
confirmed, CBEMED will make a refund upon proof
of value.

The reseller must complete:
Name: _____
Address: _____
Length: _____
ZIP CODE: _____
City: _____
UF: _____
Invoice No: _____

Date: ____/____/____
This warranty term must be filled out by the dealer
at the time of sale. If there is a manufacturing
defect, it must be presented together with the
purchase invoice.

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Congratulations on the purchase of your new BIC Stethoscope

GENERAL

The BIC Stethoscope is an instrument indicated for non-invasive auscultation of cardio-respiratory noises internal to the organism. The sounds are obtained by contacting the diaphragm with the region of interest. These

sounds, obtained through the diaphragm, to the operator ear. The binaural earphone allows you to hear the sound simultaneously with both ears.

CONSERVATION AND STORAGE

- Protect the stethoscope from water, moisture, impacts, falls, sunlight, heat and fire.
- Clean the stethoscope with a soft cloth, slightly moistened with water, neutral or 70% alcohol. The use of corrosive products, detergents or polishers is prohibited.
- The tube must not deform, be stretched, have sharp edges or have contact with sharp materials.
- Do not drop it and avoid shocks.
- Recommended storage temperature: - 20 ° C to 45 ° C.
- Recommended relative air humidity: maximum 80%.

HOWTO USE


- Standard Adult, Pediatric Model and Neonatal:
- Fit the binaural earphone in your ears in order to isolate all external sounds.
- Touch the diaphragm to the area to be analyzed.
- Lightly touch the membrane a few times to see if the eartips are positioned correctly in the ears, this will produce a noise.

Adult Double Model, Pediatric Double, Neonatal Double, and Cardiological Model:

- Fit the binaural earphone in your ears in order to isolate all external sounds.
- Lightly touch the membrane a few times to see if the eartips are positioned correctly in the ears, and which diaphragm is in the position of use, this will produce a noise.
- Touch the diaphragm to the area to be analyzed.
- To change the diaphragm to be used (cardiac or pulmonary side), hold the head of the diaphragm/headphone and rotate it 180° (degrees).


COMPOSITION AND FORMS OF PRESENTATION
Standard model

- The Standard type stethoscope has a type of headphone:
- Simple closed headphone with diaphragm.
- 1) Angle in chromed metal and / or treated
- 2) PVC "Y" tube
- 3) Pair of PVC and / or silicone Olives (depending on model)
- 4) Steel spring
- 5) Headphone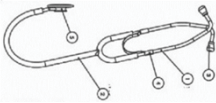

Double Model

- The double model stethoscope has two types of headphones:
- Closed with rigid plastic membrane (diaphragm): used to detect low frequency sounds with greater distinction and precision.
- According to the model, the pulmonary side can also be opened, without the membrane (bell-like).
- To change the diaphragm to be used (cardiac or pulmonary side), hold the head of the diaphragm/headphone and rotate it 180° (degrees).
- 1) Angle in chromed metal and / or treated
- 2) PVC "Y" tube
- 3) Pair of PVC and / or silicone Olives (depending on model)
- 4) Steel spring
- 5) Headphone with Isolating Ring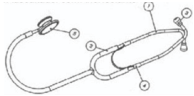


Cardiological Model

- The double model stethoscope has two types of headphones:
- Closed with rigid plastic membrane (diaphragm): used to detect low frequency sounds with greater distinction and precision.
- According to the model, the pulmonary side can also be opened, without the membrane (bell-like).
- To change the diaphragm to be used (cardiac or pulmonary side), hold the head of the diaphragm/headphone and rotate it 180° (degrees).
- 1) Stainless steel angle
- 2) PVC "Y" tube
- 3) Pair of PVC and / or silicone Olives (depending on model)
- 4) Steel spring
- 5) Matte stainless steel earpiece
- a) Threaded matte stainless steel base
- b) PVC and / or resin / fiber membrane
- c) PVC ring

Follow:

- A pair of threadable Olives;
- APVC and / or resin / fiber membrane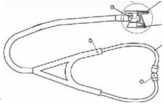

WARRANTY

Your BIC product, when new, is guaranteed against original manufacturing defects. The warranty period starts on the date of purchase of the factory or its authorized distributors. CBEMED's obligation is limited to the repair or replacement of components found to be defective by CBEMED within the warranty period. These warranties apply to the original purchaser and cannot be distributed or transferred to third parties. This warranty does not apply to any damage or failure of the product considered by CBEMED, in its analysis, to be due to misuse, accidents (including damage to transport), falls, neglect, improper maintenance, violation, modification or repair by persons unauthorized. These express warranties take precedence over any and all other warranties, express or implied, including warranties of merchantability and fitness for a particular purpose, and no other person is authorized to assume on BIC's behalf any other warranties connected with the sale of the product. CBEMED is not responsible for any loss or damage, whether direct, incidental or consequential, resulting from the breach of any express warranty, except as provided herein. The manufacturer reserves the right to replace the defective product with a new one, if deemed necessary, being the sole judgment of the manufacturer. Repairs carried out within the warranty period do not extend the warranty period.

- PRO EFFICACE LINE: 3 years warranty against manufacturing defects;
- ETERNITY LINE: 5 years warranty against manufacturing defects;
- BIC PROFESSIONAL LINE: 5 years warranty against manufacturing defects;
- CARDIOLOGIC LINE: 5 years warranty against manufacturing defects.